



- 1 Go to www.nutridapt.com and click **Log In**

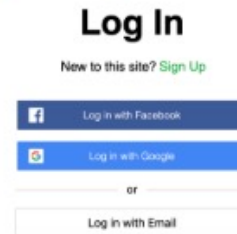


- 2 Click on **Sign Up** and then register using the Clinic ID supplied by Canine Biologics. We advise against using a shared email address.




- 3 Click the **For Clinicians** menu, then **Add Patient**. Enter the patient information. Use existing **EMR ID** for the Patient ID field.

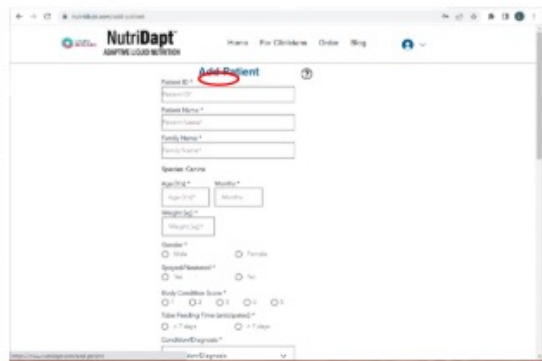

- 4 Review Recommended %RER, Protein and Fat and adjust if needed. Click **Save** to view the patient's recipe.

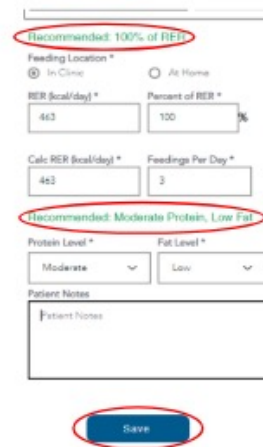

- 5 Using the measurements provided, mix and feed! Print, if desired, using your browser's print feature.

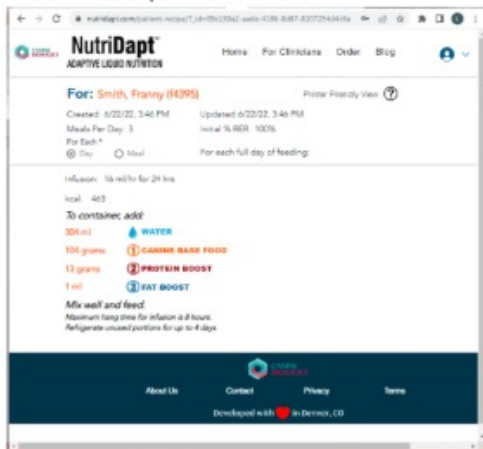

- 6 Use **View Patients** and **Update Patient** to find and update a previously entered patient record and get an updated recipe. Refer to the **Help** and **FAQ** pages for more detailed information and additional features.

That's all there is to it. Go from entering a patient to feeding in under five minutes!